




VersaMed iVent 201 Power Pack Replacement

This guide will allow users to disassemble and reassemble the Power Pack on the VersaMed iVent 201.

Written By: Jeffrey Yin

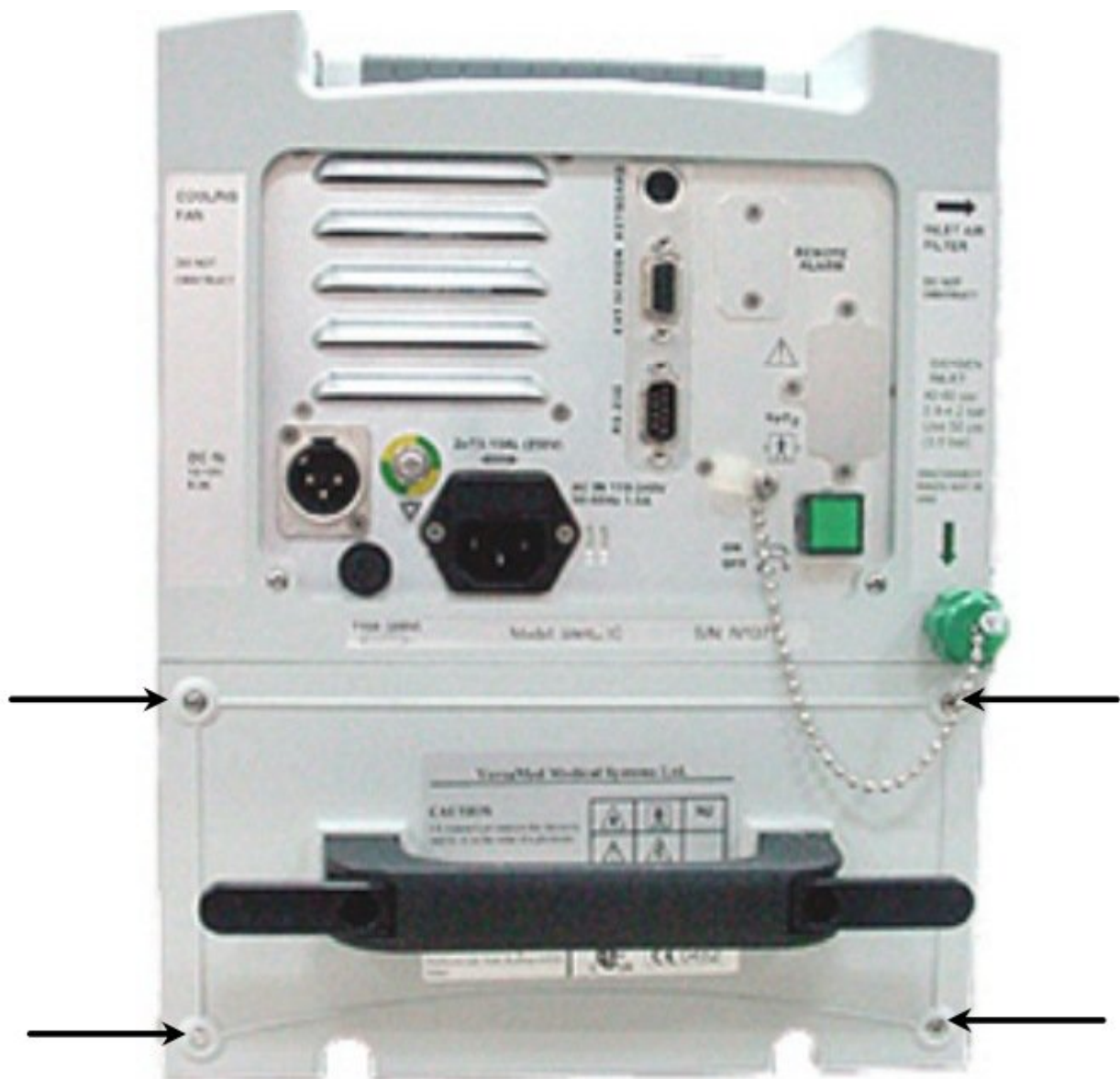

INTRODUCTION

This guide provides detailed steps on replacing your device's power pack. For more information on replacing the power pack, please refer to **page 144** in the service manual.



TOOLS:


- [Phillips #1 Screwdriver](#) (1)
- [Large Needle Nose Pliers](#) (1)



PARTS:


- [VersaMed iVent 201 Power Pack](#) (1)

Step 1 — Power Pack

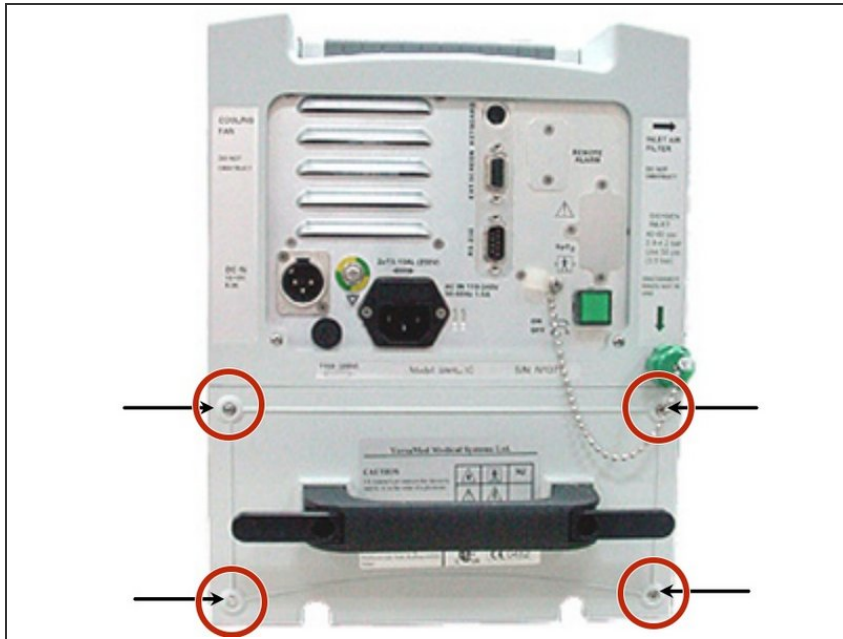


- Remove the four 3mm x 20mm Phillips screws located on the corners of the power pack.

Step 2

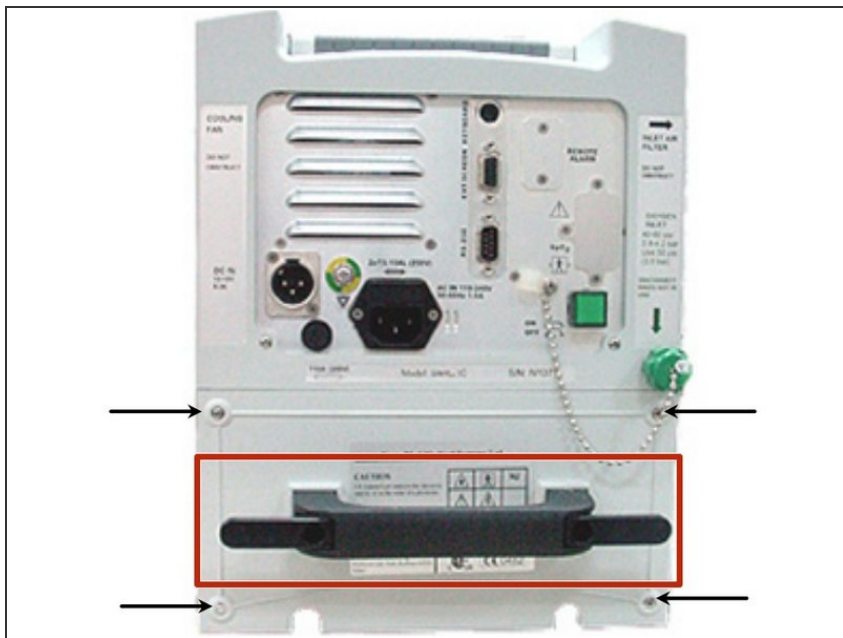


- Pull the power pack from the compartment using black handle.

Step 3

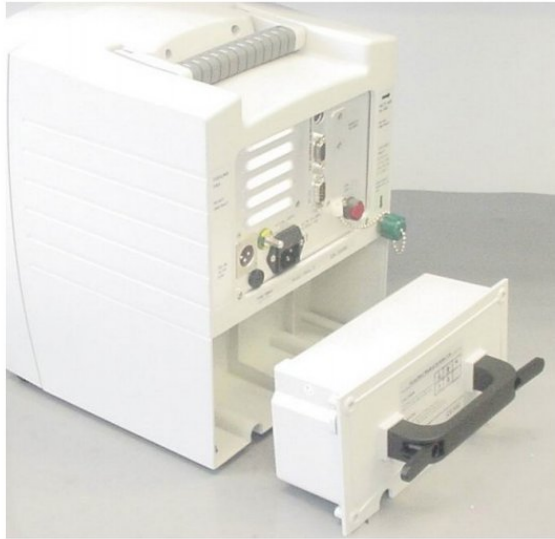


- Remove the five 3mm x 10mm Phillips screws and washers from the back of the power pack. Lift off the cover after.

Step 4

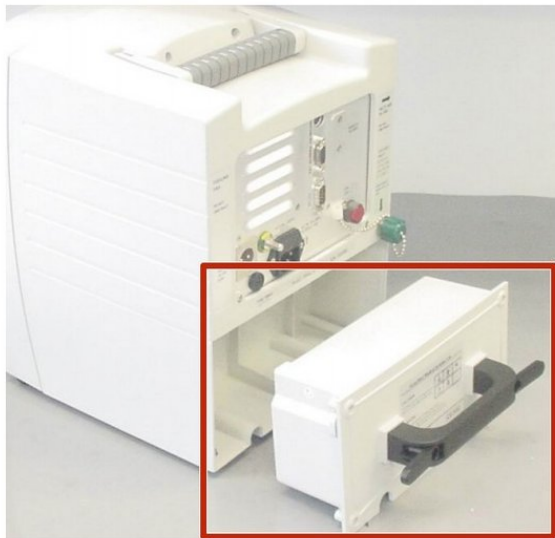

- Turn the battery pack upside down and take the battery out of the casing.
- ⓘ When reassembling, listen for the sound of the cooling fan for confirmation that the connection was successful.

Step 5

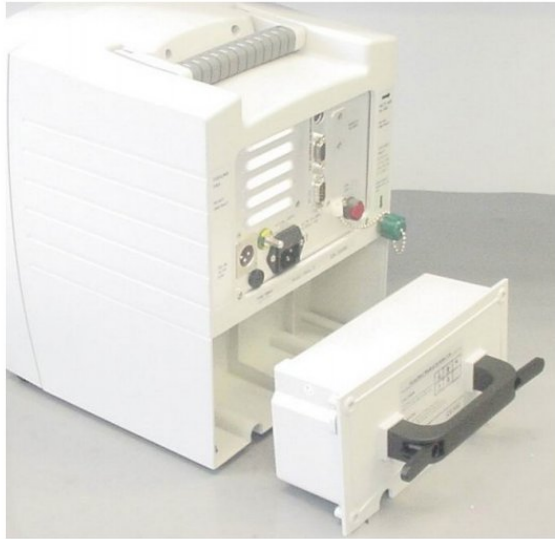

- Remove the two spade connectors using pliers.

⚠ When assembling the power pack, **MAKE SURE** to connect the red wire to the positive terminal and the black wire to the negative terminal of the battery.

To reassemble your device, follow these instructions in reverse order. **Caution: MAKE SURE THE RED WIRE IS CONNECTED TO THE POSITIVE TERMINAL AND THE BLACK WIRE IS CONNECTED TO THE NEGATIVE.**